# LABOLIGHT LV-III

## LABOLIGHT LV-III

### LIGHT CURING UNIT

For use only by a dental professional as recommended in these instructions.

#### RECOMMENDED USE

Laboratory based curing of visible light-cured dental composites, base materials, etc.

#### CAUTION

- Indoor use only.
- Only use an electrical outlet with ground (earth) connection.
- Locate the unit in a clean, well ventilated, vibration free area away from direct sunlight or other sources of heat.
- When cleaning or changing parts always unplug the unit.
- Ensure that the door is fully closed when the fluorescent lamps are illuminated.
- Do not look directly at the fluorescent lamps while they are illuminated.
- Do not touch the fluorescent lamps during use or for a time after turning off to prevent burns.

#### IDENTIFICATION OF PARTS

- Door
- Window
- Main Lamp  
The main lamp will go on and off (flash) at the time of total 200 hour irradiation of the fluorescent lamps indicating time for lamp replacement.
- Control Box
- Main Switch  
With the main switch on, the main lamp illuminates so that each part can function.
- Stop Key  
The stop key is pushed to stop irradiation in process. The fluorescent lamps will not function.
- Curing Key  
Irradiation time (0.5 min - 1 min - 3 min - 5 min.) can be selected by pushing each key. The fluorescent lamps illuminate when the curing key is pushed. The lamps turn off when the selected time is reached.
- Time Lamp  
With the curing key on, the timer lamp illuminates and indicates the selected irradiation time.
- Fixation Screws for the curing case
- Fan  
Maintains correct temperature inside the curing chamber.
- Fuse x 2 pieces
- Reset Switch / Time Counter (built-in)  
After replacing the fluorescent lamps, push the reset switch to reset the time counter.
- Power Cord
- Fluorescent Lamp
- Temperature Sensor
- Thermostat
- Reflector
- Curing Case
- Turntable  
The turntable rotates when the fluorescent lamps illuminate.
- Sliding Stand
- Fluorescent Lamp (3 sets)
- Vinyl Cover (1 piece)
- Rest (2 pieces)
- Small Post (8 pieces)
- Large Post (8 pieces)
- Turntable (2 pieces)

#### PREPARATIONS

The fluorescent lamps and turntable come separate from the main body for safety during transportation. Mount them as follows.

- Fixation of Fluorescent Lamps**  
1)Remove the 4 fixation screws on the curing case. Pull out the case towards Sicherheitsgrundseparat verpackt. Sie werden wie folgt montiert:  
2)Insert the lamp into the lamp socket in the direction of Arrow 1 and hold. Turn gently in the direction of Arrow 2 to fix the lamp (Photo 1).  
3)Insert the curing case at the front. Fix with the 4 screws.  
Note: When fixing or replacing the fluorescent lamps, be careful not to touch the sensor or thermostat. This may result in a breakdown of the unit.

- Fixation of Turntable**  
Open the door and pull out the sliding stand to fix the turntable.
- Installation of Unit**  
Install the unit more than 5 cm (2 inches) away from the wall. The cooling fan at the rear takes in air to maintain correct temperature inside the curing chamber.

#### DIRECTIONS FOR USE

- Turn on the main switch. The main lamp illuminates.
- Open the door to slide the turntable out. Place the resin prosthesis on it (Photo 3).
- Slide the turntable back fully into the curing case and close the door.
- Push the curing key according to the curing time required for the resin prosthesis. The timer lamp illuminates and irradiation starts. To change the irradiation time in progress, push another curing key. The new selected time will be added to the elapsed time until the previously selected time. For example, if the minute key was pushed first and then after 20 seconds the minute key is pushed, the irradiation time will be changed to 5 minutes and 20 seconds. Notes:  
1)If the fluorescent lamps are cold, they may not function properly. If room temperature is below 15°C (59°F), do a trial irradiation for 3 minutes before using the resin prosthesis.  
2)To set the correct irradiation time, follow the manufacturer's directions for use of the light-cured resin.
- When the irradiation time is reached, the fluorescent lamps will go out. Open the door to slide the turntable out. Remove the resin prosthesis.

#### REPLACEMENT OF LAMPS / RESET OF TIME COUNTER

The total effective irradiation lifetime of the fluorescent lamps is 200 hours. The cumulative irradiation time is recorded by the built-in time counter inside the unit. When the main lamp goes on and off (or flashes), replace all 3 lamps. Always use the recommended GC special lamp for the GC LABOLIGHT LV-II or LV-III.

- A. Lamp replacement**  
1. Turn off the main switch.  
2. Unplug the unit.  
3. Make sure the lamps to be replaced are cold.  
4. Remove the 4 fixation screws on the curing case and pull out the case (Photo 4).  
5. Push the lamp gently in the direction of Arrow 1 and hold. Turn in the direction of Arrow 2 to pull the lamp off (Photo 4).  
6. Replace the lamps, referring to "Fixation of Fluorescent Lamps" mentioned as above (Photo 5).  
7. Insert the curing case at the front. Fix with the 4 screws.
- B. Reset of Time Counter after lamp replacement**  
1. Make sure the main switch is on. If it is off, you cannot reset the time counter.  
2. Push a reset switch on the back of the unit (Photo 6). The flickering (flashing) main lamp will stay on.

#### REPLACEMENT OF FUSE

- When the fuse is blown,  
1. Turn off the main switch.  
2. Insert a "minus" (or straight-tipped) screwdriver in a little upward position into the opening of the fuse box on the back of the unit.  
3. Remove the fuse by softly lifting the screwdriver handle up (Photo 7).

#### CARE OF THE LABOLIGHT LV-III

Do not attempt to modify the unit. When cleaning the unit, always switch off and unplug the mains. Check the unit periodically to ensure satisfactory operation.

Clean the fluorescent lamps, reflectors and the unit exterior once a month. Clean the lamps and reflectors whenever they are dirty, otherwise curing performance may be affected.

- To clean the fluorescent lamps, reflectors and unit exterior. Soak a soft cloth in lukewarm water containing a little detergent, wring well and wipe. Avoid using strong solvents such as thinner or petroleum benzine.
- To clean the turntable Soak a soft cloth in lukewarm water containing a little detergent, wring well and wipe. If disinfection is required, wipe gently with a cloth soaked in medicated liquid, wash with water and dry well.

#### TROUBLESHOOTING

In case of difficulty, check the following before seeking repair.

Symptom	Check	Correction
The main lamp does not come on even if the main switch is turned on.	Check if the plug is disconnected.	Plug in correctly.
Check if the fuse is blown.	Replace the fuse.	
The fluorescent lamps do not come on even if the curing key is off.	Check if the main switch is off.	Turn on the main switch.
Not all the fluorescent lamps illuminate.	Check if the lamps are set into the lamp sockets correctly.	Set the lamps into the lamp sockets correctly.

#### WARRANTY / REPAIR

Warranty year : 1 year from the date of purchase. In case of a breakdown during the term, repair will be carried out free of charge provided that the unit has been used under normal conditions and according to the instructions for use.

Note: Consumables (such as lamps) are not warranted.

#### SPECIFICATIONS

Power supply	AC120V 60Hz	AC220 / 230V 50Hz
Power consumption	120W 1.6A	120W 0.9A
Lamp rating	27W x 3 pieces	
Fuse	5A x 2 pieces	2A x 2 pieces
Effective turntable space	90mm (dia.) x 70 mm (H)	
Effective room temperature	10 - 30°C (50 - 86°F)	
Dimensions	185mm (W) x 245mm (D) x 275mm (H)	
Weight	10 kgs.	

Specifications are subject to change.

The technical values can change.

Prior to use, carefully read the instructions for use.

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